



Skin Barrier Repair Medical Devices



For use after medical aesthetic procedures, on sensitive skin, or in areas where the skin barrier is compromised.

Changsha Hairun Biotechnology Co.,Ltd

Restoration of the skin barrier after non-surgical or minimally invasive aesthetic procedures

Let's take a look at which superficial and deep medical aesthetic procedures these products are suitable for:

Select recommended aftercare products based on the condition and needs of your skin (preference should be given to medical devices).



	Superficial aesthetic procedures	Deep aesthetic procedures
Main functions	Addresses epidermal concerns such as dull complexion, superficial pigmentation, and fine lines, with a focus on daily care and early anti-aging treatment.	Aimed at addressing skin concerns, including deep wrinkles, severe acne scars, scar tissue, and skin laxity, with a focus on deep restoration and advanced anti-aging care.
General procedures	Photorejuvenation, glycolic peeling, microneedling therapy, moisturizing injections, hyaluronic acid injections, Botox injections.	Fractional laser (ablative/non-ablative), picosecond laser, Q-switched laser, microneedle therapy using gold, Thermage, Ultherapy.
Recovery period	Short, usually 1-3 days for full recovery; some procedures (such as microneedling) may require 3-5 days.	A longer recovery period is required for ablative fractional laser procedures (7-14 days), non-ablative procedures (3-7 days), and gold microneedle therapy (approximately 5-7 days).
Suitable for	People with generally healthy skin who have minor problems or need regular care.	People with more pronounced skin problems, such as deep wrinkles, acne scars, noticeable scars, or significant skin laxity.
Key points of postoperative care	Pay special attention to moisturizing and sun protection. Use hyaluronic acid-based medical products to restore your skin and avoid irritating cosmetics.	Strict sun protection is essential. Use medical products based on recombinant collagen to prevent infection and hyperpigmentation.


Emergency Hydration, Barrier Repair, Soothing and Calming, Intensive Nourishment

Medical Sodium Hyaluronate Dressing

MEDICAL SODIUM HYALURONATE DRESSING is composed of Sodium Hyaluronate Solution, Glycerin, Carbomer, D-(+)-Trehalose Dihydrate, Triethanolamine and Purified Water and Non-woven Dressing. Non-sterile product. The essence is colorless to light yellow liquid, The essence amount shall not be less than 95% of the marked packing amount, the total packing amount shall not be less than the marked packing amount.

HOW TO USE

1. For external use only.
2. Clean the affected skin, open the outer package, take out the MEDICAL SODIUM HYALURONATE DRESSING, spread it out, apply it on the facial skin, gently press the dressing to the skin.
3. Use once a day for 15 minutes for 2 weeks, then ask your doctor for advice.

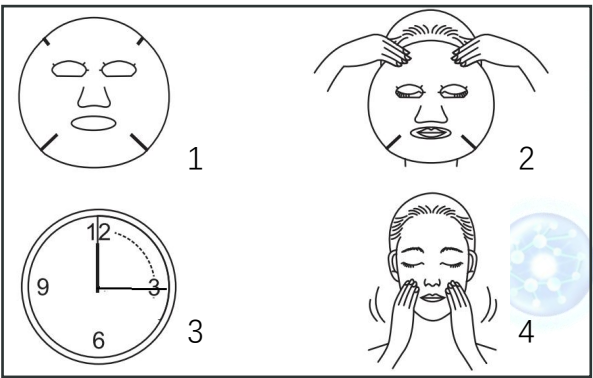

- MEDICAL SODIUM HYALURONATE DRESSING is intended for age 22-60 people who need to skin repair, to provide a moist environment to relieve symptoms of redness, itching, tingling, warm, dryness and desquamation caused by injured skin or mucous membrane, dermatitis, skin care after cosmetic surgery, aesthetic photonics surgery.

Shelf life is 2 years. Sterilized by irradiation. Store in a dry place. Protect from light.

Order no.	Shape	Package	Remark
P-67-1	25g / For Face	5 sheets/box, 60 boxes/carton	Fabric masks
P-67-2	28g / For Face	5 sheets/box, 60 boxes/carton	Fabric masks
P-67-3	8g / For Eye	10 sheets/box, 60 boxes/carton	Eye mask


Physical Protection, Wound Healing, Anti-inflammatory and Redness Reduction, Long-lasting Moisturization

Medical Hyaluronic Acid Hydro Gel

This product contains glycerin, sodium hyaluronate solution, D-(+)-trehalose dihydrate, carbomer, triethanolamine, and purified water.

INSTRUCTIONS FOR USE

1. Adjuvant therapy for minimally invasive laser facial surgery, mild to moderate inflammatory acne, dermatitis and eczema, sensitive skin, and hormone-dependent dermatitis. Apply the gel externally to the affected areas after cleansing. Use twice a day or more often.
2. For childhood eczema, nodular urticaria, and scar repair, apply the gel externally to the affected areas. Use twice a day or more often.
3. Daily skin care: apply evenly to the affected areas after cleansing. Use twice a day.

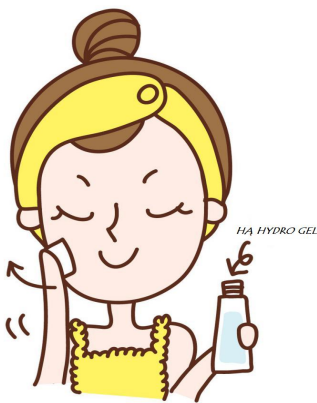

- Forms a transparent protective layer on the skin, relieves inflammation, redness, itching, tingling, heat, dryness, and flaking caused by acne, dermatitis, eczema, and hormone-dependent dermatitis.
- Suitable for treating acne, skin allergies, and closed wounds caused by laser cosmetic surgery, photorejuvenation, and other minimally invasive procedures.
- Also suitable for restoring and caring for normal skin, especially sensitive skin.

Shelf life is 2 years. Sterilized by irradiation. Store in a dry place. Protect from light.

Order no.	Size	Package	Remark
P-52-2	20g	100 pcs per carton	

Deeply Penetrating Repair, Daily Stability Maintenance, Soothing Sensitivity, Promoting Regeneration

Medical Hyaluronic Acid Skin Repair Lotion

Medical lotion for skin restoration with hyaluronic acid consists of glycerin, sodium hyaluronate solution, carbomer, D-(+)-trehalose dihydrate, triethanolamine, and purified water.

INSTRUCTIONS FOR USE

- 1. After cleansing, while the skin is still damp, apply a few drops or spray onto the face with your palms.
- 2. Use 2-3 times a day or as recommended by your doctor.
- 3. Also suitable for people with normal skin.



- Relieves skin inflammation, redness, itching, tingling, heat, dry flaking, and other symptoms caused by acne, dermatitis, eczema, and hormone-dependent dermatitis. Suitable for sensitive skin, laser phototherapy, and other minimally invasive procedures, as well as for the care of undamaged skin.

Shelf life is 2 years. Sterilized by irradiation. Store in a dry place. Protect from light.

Order no.	Size	Package	Remark
P-51-2	30ml	100 bottle per carton	

Instant Repair, Soothing & Calming

Medical Recombinant Collagen Skin Care Dressing

The product consists of a solution of recombinant collagen protein and non-woven material sealed in an aluminum-plastic bag. Recombinant collagen serum contains recombinant collagen, glycerin, carbomer, triethanolamine, xanthan gum, disodium ethylenediaminetetraacetate, sodium methylparaben, sodium propylparaben, and purified water. The product is radiation sterilized and delivered in a sterile state.



INSTRUCTIONS FOR USE

1. For superficial wounds, after laser/phototherapy/AHA peeling/minor cosmetic procedures: unfold the dressing and apply directly to the affected area. Use 1-2 times a day or increase the frequency depending on the condition of the wound. Leave on for 10-30 minutes after application.
 2. For other non-chronic wounds: clean the affected area and surrounding healthy skin. Unfold the dressing and apply it to the wound surface. Change the dressing depending on the healing process.
- For the care of non-chronic wounds (such as superficial wounds, mechanical injuries, minor cuts, first-degree burns or superficial second-degree burns, as well as wounds after laser/phototherapy/chemical peeling/minor cosmetic procedures) and surrounding skin, creating a microenvironment that promotes wound healing.

Shelf life is 2 years. Sterilized by irradiation. Store in a dry place. Protect from light.

Order no.	Shape	Package	Remark
P-87-21	28g/For face	5 sheets/box, 60 boxes/carton	Fabric masks

Physical Protection, Wound Healing

Medical Recombinant Collagen Skin Care Hydrogel

The product consists of recombinant collagen gel and a packaging container. The recombinant collagen gel consists of recombinant collagen, glycerin, carbomer, triethanolamine, sodium methylparaben, sodium propylparaben, and purified water. The product is supplied in a sterile form after irradiation sterilization.



INSTRUCTIONS FOR USE

1. Clean the affected area of skin, then apply the required amount of the product.

2. Use twice a day or more often if necessary.
- For the care of non-chronic wounds (such as superficial wounds, mechanical injuries, minor cuts, first-degree burns or superficial second-degree burns, as well as wounds after laser/phototherapy/chemical peeling/minor cosmetic procedures) and surrounding skin, creating a microenvironment that promotes wound healing.

Shelf life is 2 years. Sterilized by irradiation. Store in a dry place. Protect from light.

Order no.	Size	Package	Remark
P-89-21	20g	100 bottle per carton	
P-89-22	25g	100 bottle per carton	
P-89-23	30g	100 bottle per carton	

Deep Repair, Daily Stabilization

Medical Recombinant Collagen Skin Care Solution

The product consists of recombinant collagen gel and a packaging container. The recombinant collagen gel consists of recombinant collagen, glycerin, carbomer, triethanolamine, sodium methylparaben, sodium propylparaben, and purified water. The product is supplied in a sterile form after irradiation sterilization.



INSTRUCTIONS FOR USE

1. Clean the affected area and surrounding healthy skin.

2. Apply topically by spraying 2-3 times a day or as directed by your doctor.
- For the care of non-chronic wounds (such as superficial wounds, mechanical injuries, minor cuts, first-degree burns or superficial second-degree burns, as well as wounds after laser/phototherapy/chemical peeling/minor cosmetic procedures) and surrounding skin, creating a microenvironment that promotes wound healing.

Shelf life is 2 years. Sterilized by irradiation. Store in a dry place. Protect from light.

Order no.	Size	Package	Remark
P-88-21	5ml	100 bottle per carton	
P-88-22	50ml	100 bottle per carton	



ALL FOR PEOPLE'S HEALTH

Changsha Hairun Biotechnology Co., Ltd.

Contact persons: Hou Lun / Liu Jiun

Enquiries via WhatsApp: <https://wa.me/8617788969619> or
<https://wa.me/8619173969821>

Mobile phone: +86-17788969619 / +86-19173969821

WeChat: 19173969821

Email: helenhouhr@foxmail.com / monument1717@gmail.com

Address: 5-8 Floor, A4 International Medical Instrument Industrial Park, Hunan
Lugu Road, No. 229, Valley Road, Changsha Hi-tech Development Zone, Changsha,
Hunan, China Official website: <http://hairunbiotech.com/>